

**Island Home Medical
John Curelli, RRT**

(508)693-4380

Nebulizer Instructions

Purpose To provide for aerosol nebulization of inhaled medications.

Procedure Assemble nebulizer.
Connect tubing to outlet port.
Add prescribed medicine to aerosol chamber.
Turn power "ON"
Nebulizer should be held upright.

Observe adverse reaction
precautions of medication.

Turn power "OFF" when medicine no longer mists.

Cleaning After each treatment: disassemble nebulizer, disconnect from tubing, soak parts in warm water and a liquid dish soap.
Rinse well, air dry.

Refer to the included manufacturer's complete instruction booklet.

Service *For equipment service questions or replacement:*
Call 508-693-4380