

Island Home Medical
12 Old Schoolhouse Village
Vineyard Haven, MA 02568
24 Hours: 508-693-4380

Instructions Portable Oxygen Tank with Conserver

**The tank will provide oxygen for a total of ____ hours at
____ liters flow.**

****No Smoking No Open Flame****

Procedure

- Attach oxygen tubing to regulator.
- Place wrench onto tank stem

To Open

- Turn wrench on top of tank ½ turn counter clockwise. The amount of oxygen remaining in the tank will read on the gauge. *Check contents of tank before each use.*
- Set flow on dial to ____ liters. You will feel and hear a slight puff with each breath.

To Close

- Turn wrench clockwise until snug.
- Turn black dial to 0.

To Replace Empty Tank

- Close tank.
- Turn “T” handle on regulator counter clockwise until you are able to lift regulator upwards.
- Remove white plastic cap off full tank.
- Slide regulator over top of tank, line up, and then insert 2 pins into corresponding holes on tank.
- Turn “T” handle clockwise until it is snug.

For replacement tanks call John Curelli @ 508-693-4380.

Please call at least 24 hours in advance to order tanks.

Place orders on Thursday for a Friday delivery so that you have enough tanks for the weekend.